

Raw

c o n d o m s

HOW TO USE THE CONDOM

- ! Verify the expiry date on the condom wrapper before using it.

Put the condom on the erect penis before any contact with your partner (helps to prevent pregnancy and the possibility of catching sexually transmitted infections).

- 1 Tear the wrapper from the serrated edge and handle the condom carefully, as sharp objects like nails, teeth or jewelry can damage.

- 2 Make sure the roll is on the outside. Squeeze the teat end of the condom to avoid air trapped inside.

- 3 Place the condom on top of the erect penis, still squeezing the teat, roll it down with your other hand. If it comes off or breaks, stop and put a new one.
- 4 After the sexual act, while the penis is still erect, the condom should be held firmly in place at the base of the penis before withdrawal. Then take the condom off, wrap it in a tissue and throw it in the bin.

THE RAW CONDOM

Is a method of contraceptive electronically tested to the effects to help to reduce the risk of pregnancy and catching sexually transmitted infections. Attention: No method of contraception can provide 100% protection against pregnancy or sexually transmitted infections. Choosing correctly the condom size helps to reduce the risk of pregnancy and catching sexually transmitted infections.

Is made of natural latex.

It is not recommended to people allergic to latex. Consult a doctor before using it.

Contains added lubricant gel which does not influence the sperm.

If you want to add other oils take care to only use lubricants that are recommended for use with condoms. The use of other oil-based ones as petroleum jelly, massage oil, vaseline or shower gel can damage condoms.

For anal sex, it is advised the use of additional lubricant on the outside of the condom to help to decrease the risk of the condom breaking or coming off.

Is for one time use.

A new condom should be used for any other type of sex to be practiced. Once used, wrap it in a tissue and throw it in the bin. Please do not flush it down the toilet in order to protect the environment.

Should be stored in a cool (<25°C) dry place away from direct sunlight.

Raw
c o n d o m s

CE
0123



Designed by:
Ana Mirats Studio
Barcelona, Spain.

Distributed by:
Health Protect S.L.
Madrid, Spain

Represented in EU by:
Shanghai International
Holding CorpGmbH
(Europe)
Eiffestraße 80
20537 Hamburg,
Germany

Produced by:
DongYang Songpu Latex
(Jinzhou) Co., Ltd.
New and High-tech
development Zone,
121000 Jinzhou,
Liaoning Province, PRC



rawcondoms.com

